


INSTRUCTIONS FOR OPERATION AND CARE OF

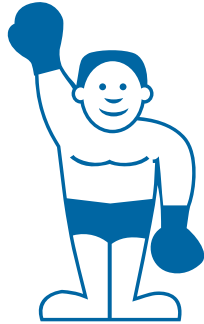

LITTLE
freeze

Compact Chilling Unit

Model LF-4



Whitehall
Manufacturing[™]

P.O. Box 3527 • City of Industry, CA, 91744-0527, U.S.A.

800-782-7706 • 626-968-6681 • www.whitehallmfg.com

Member of  MORRIS GROUP
INTERNATIONAL



Introduction 3
Installation And Operation 3
Troubleshooting 4
Parts List And Diagram 4

Instructions for Operation and Care of Little Freeze Compact Chilling Unit



Congratulations on receiving your compact Little Freeze Compact Chilling Unit.

The Little Freeze provides a convenient and reliable storage for all Glacier Packs.

The unit comes standard with specially designed shelves that facilitate the easy removal of the four included Glacier Packs. The bottom shelf has spaces for four ice cups (not included).





INSTALLATION AND OPERATION

Place your appliance in a location that is strong enough to support its total weight, keeping in mind that it will weigh substantially more when fully loaded than when empty.

In order to prevent vibration and noise, the refrigerator should be leveled by adjusting the front leveling legs.

Please note that this appliance is designed for “free-standing” use only. Choose a well ventilated location with at least 2-3 inches of clearance around the appliance.

Locate the appliance out of direct sunlight and away from heat sources.



WARNING- This appliance must be installed on a circuit that is grounded.

Operation

Before plugging in your freezer for the first time remove all packaging, clean the freezer with a soft rag and a mixture of lukewarm water and baking soda (2 tablespoons baking soda in 1 quart of water works well). Wipe dry with a soft cloth.

Temperature Control

The temperature in the appliance is controlled by the thermostat control knob located in the lower section. Turn the thermostat control knob to regulate the temperature inside the unit. The knob has markings to indicate the relative temperature settings, but in general, turning the knob clockwise corresponds to warmer temperatures: turning the knob clockwise corresponds to colder temperatures.

Please note that this freezer has a full temperature range, allowing you to select refrigerating or freezing temperatures.

Lock

This freezer ships with a lock and two keys. To lock the door, close the door, insert the key into the lock, and turn. To unlock the door, insert the key into the lock and turn, then open the door.

Defrosting

You may defrost this freezer once every few months or whenever 1/4” of frost develops on the walls.

Remove the electrical plug from the wall socket, open the door and remove all contents. Defrost using a plastic scraper, or let the temperature rise naturally until the frost melts, then wipe away the remaining ice and water.

You may also want to clean the unit at this time. Once you have finished, plug the freezer back in and make sure the

temperature knob is still set where you would like it to be set.

The colder the settings, the more frost will build up. Additionally, the more humid the ambient conditions, the more frost will build up. Keep the doors closed as much as possible and avoid opening the door unless necessary to minimize frost build-up.

Cleaning Your Freezer

Before you start to clean your freezer, remove the electrical plug from the power supply. You may want to defrost the freezer at this time. If so, follow the above instructions.

To clean the freezer, use a soft rag and a mixture of lukewarm water and baking soda (2 tablespoons baking soda in 1 quart of water works well). Clean the freezer and then dry it with a soft cloth.

Storage

If the freezer is not in use for a long period of time, it should be stored.

Disconnect the freezer from the power supply by removing the plug from the wall socket.

Defrost, clean and dry inside.

Leave the door open to prevent any unpleasant smells from building up while the freezer is not in use.

TROUBLESHOOTING

Please check the following points:

The freezer is not working

- Check that the plug is inserted correctly into the wall socket.
- Check that there is power to the wall socket by plugging in another appliance.
- Check the fuse.

The freezer is noisy when running

- Check to make sure that the freezer is level and is not in contact with another appliance or furniture.
- Note: a gentle trickling sound is normal; this is likely coolant in the sealed system or condensate water from the unit.


The freezer does not cool sufficiently

- If the door has been opened too often, or it has been left open for a while, it will take time for the freezer to reach it's set temperature.
- Check that the airflow from the rear of the freezer has not been reduced or obstructed due to insufficient clearance.



PARTS LIST FOR LITTLE FREEZE

ITEM #	PART #	DESCRIPTION
1	6702-450-000	Standard Size Glacier Pack
2	6702-494-001	Rack Assembly
3	6702-486-000	Thermometer
4	6702-487-199	Cup Holder (Ice Cups not included)

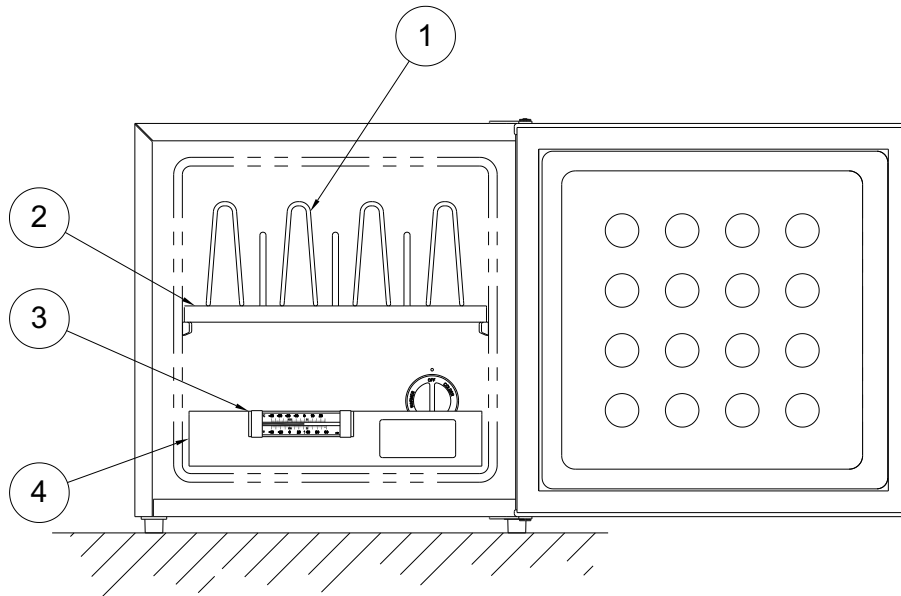



Mailing Address:

P.O. Box 3527 • City of Industry, CA 91744-0527 U.S.A

Physical Address:

15125 Proctor Avenue • City of Industry, CA 91746 U.S.A

Phone 800-782-7706 • 626-968-6681

Fax 626-855-4862

Web: www.whitehallmfg.com

E-mail: info@whitehallmfg.com

Any questions regarding servicing of products in Europe or the Middle East should be directed to:
Niva Medical Company • Olarinluoma 14 B, FIN-02200 Espoo, Finland
Phone 358 9 615 00 540 • Fax 358 9 615 00 545
Web www.nivamedical.fi • E-mail info@nivamedical.fi

MANUFACTURER'S WARRANTY

Whitehall Manufacturing Company warrants that its products are free from defects in material or workmanship under normal use and service for a period of one year from date of shipment. Whitehall's liability under this warranty shall be discharged solely by replacement or repair of defective material, provided Whitehall is notified in writing within one year from date of shipment, F.O.B. Industry, California.

This warranty does not cover installation or labor charges, and does not apply to materials which have been damaged by other causes such as mishandling or improper care or abnormal use. The repair or replacement of the defective materials shall constitute the sole remedy of the Buyer and the sole remedy of Whitehall under this warranty. Whitehall shall not be liable under any circumstances for incidental, consequential or direct charges caused by defects in the materials, or any delay in the repair or replacement thereof. This warranty is in lieu of all other warranties expressed or implied. Product maintenance instructions are issued with each fixture, and disregard or non-compliance with these instructions will constitute an abnormal use condition and void the warranty. If you have any questions or require technical assistance, please call 800-743-8219.

NOTICE TO KEEP ORIGINAL PACKAGING- Regarding warranty claims: customer must retain original packaging for one year upon receipt of product. If packaging is discarded, it is the customer's responsibility to provide adequate packaging. Any shipping claims that are a direct result of customer-provided packaging materials will be handled by the shipper.